## Hospital Masks – Instructions (1/5)



## Materials:

- One piece of 100% cotton, tightly woven fabric.
  Cut to the size of 15" x 7.5" (38cm x 19cm)
  NO quilted or perforated fabric
- One piece of Cut Away Embroidery Stabilizer or Interface (14" x 6.5") \*if available
- 2 heavier <u>plastic-coated wire</u>, (such as a twist tie)
  4 6 inches in length (no paper twist ties as these will not stand up to washing)
- Extra wide double fold bias tape 8 inches per mask
- Braided elastic for ear loops. Recommend 1/16 to 1/8 inch, about 20 inches



www.bit.ly/face-masks-tutorial




Wash and dry all fabric before sewing to preshrink.

	2 Cut material 7 <sup>1</sup> / <sub>2</sub> " width by 15" length	3 Iron top width of material down 1" at the top and ½" at the bottom towards the wrong side of the material.
	<u> </u>	(
	right side 15"	1" wrong side
		1/2"
1		

## Hospital Masks – Instructions (2/5)



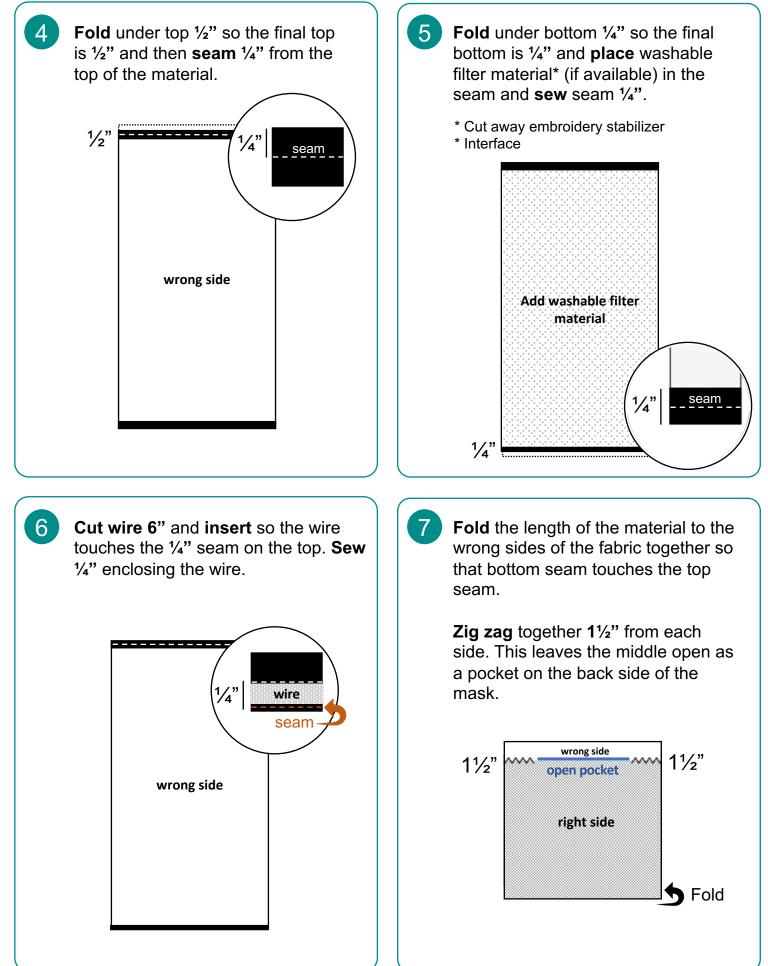


9



8 **Iron** 3 pleats on the right side of the mask and press. The first pleat should be 1½" from the top of the mask. The second pleat about ¾", third pleat ¾" and remainder of the fabric ¾". The length should be **3.75**".


Sew ¼" on each side of the mask to hold in the pleats.

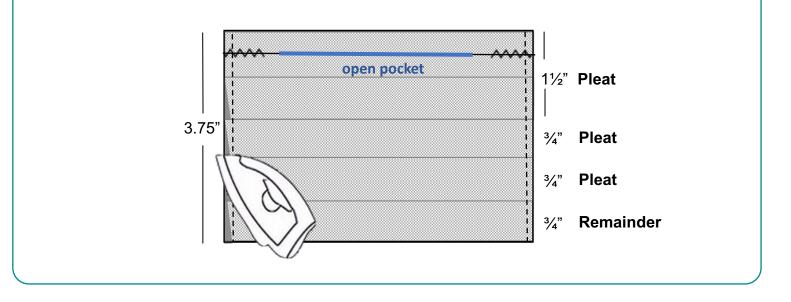

Cut a piece of fabric 2" wide by 6" long for the bind. With right sides together on the pocket side of the mask, extend  $\frac{1}{2}$ " of binding at the top of the mask and  $\frac{1}{2}$ " (or remainder of the binding) extended on the bottom of the mask.

**Fold** this 1/2" towards the non-pocket side of the mask on both the top and bottom of the mask.

**Sew** <sup>1</sup>/<sub>2</sub>" seam to incorporate the folded binding at the top and bottom.

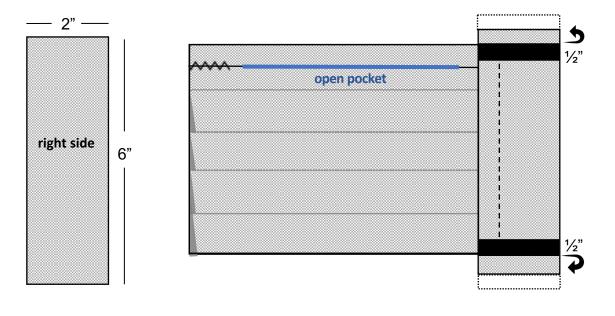


10

11



**Unfold** the seamed binding to the side of the mask and fold under  $\frac{1}{2}$ " toward the front of the mask (non-pocket) and **fold again** over the seam and top stitch along the  $\frac{1}{2}$ " stitch.

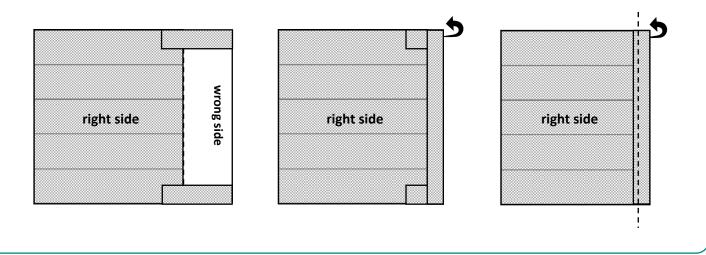

**Cut** two 10" pieces of 1/8" elastic for the ears. Thread elastic through the  $\frac{1}{2}$ " pocket created by the previous step.

Tie ends in a knot and pull knot back through to hide inside the pocket



Mask (front)

## Hospital Masks – Instructions (5/5)





Mask (back)



Mask (pocket open)